


# SingleFlowEx CD13 PE-DyLight™ 594

**ED7502**

**1 ml**

|                  |                                                                                                                                                                                                                                                                                                                                                                                                    |
|------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Description      | SingleFlowEx is fluorochrome-labeled monoclonal antibody single reagent for use in laboratory-developed multicolor flow cytometry tests in order to examine human clinical samples.<br>SingleFlowEx can be used in various multicolor combinations. Laboratory shall establish the SingleFlowEx performance characteristics in conjunction with other reagents in normal and pathological samples. |
| Clone            | WM15                                                                                                                                                                                                                                                                                                                                                                                               |
| Antigen          | CD13                                                                                                                                                                                                                                                                                                                                                                                               |
| Isotype          | Mouse IgG1                                                                                                                                                                                                                                                                                                                                                                                         |
| Format           | PE-DyLight™ 594                                                                                                                                                                                                                                                                                                                                                                                    |
| Excitation laser | blue (488 nm)                                                                                                                                                                                                                                                                                                                                                                                      |
| Concentration    | 0.080 mg/ml                                                                                                                                                                                                                                                                                                                                                                                        |
| Contents         | 1 vial of monoclonal antibody reagent.<br>The content (1 ml) is sufficient for min. 100 tests.                                                                                                                                                                                                                                                                                                     |
| Formulation      | Stabilizing phosphate buffered saline (PBS), pH 7.4, 15 mM sodium azide                                                                                                                                                                                                                                                                                                                            |
| Application      | FC (QC tested)                                                                                                                                                                                                                                                                                                                                                                                     |
| Product data     |                                                                                                                                                                                                                                                                                                                                                                                                    |

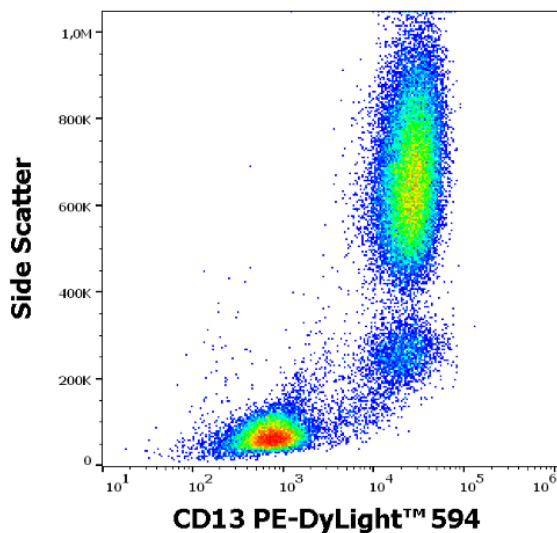

Example of surface staining pattern using SingleFlowEx CD13 PE-DyLight™ 594  
 Clone: WM15  
 Sample: EDTA-anticoagulated human peripheral blood, healthy donor  
 Staining protocol: 10 µl reagent / 100 µl blood sample; Lyse and Wash  
 Erythrocyte lysis: EXCELLYSE Easy (ED7066)

|                      |                                                                          |
|----------------------|--------------------------------------------------------------------------|
| References           | Use of this product has not been published.                              |
| Storage and handling | Store at 2-8°C. Protect from prolonged exposure to light. Do not freeze. |

Warnings and precautions

GHS Hazard Classification

Consult Safety Data Sheet (SDS) available on Product page at [www.exbio.cz](http://www.exbio.cz) for the full information on the risks posed by chemical substances and mixtures contained in the Product and how they should be handled and disposed.

Biological Hazard

Human biological samples and blood specimens and any materials coming into contact with them are always considered as infectious materials.

Use personal protective and safety equipment to avoid contact with skin, eyes and mucous membranes. Follow all applicable laws, regulations and procedures for handling and disposing of infectious materials.

Limitations of Use

Do not dilute.

Use the product directly from its original product container.

Do not use after the expiry date stated on the product labels.

Symbols

Glossary of symbols used on product packaging and labelling is available at [www.exbio.cz](http://www.exbio.cz)

The product is intended For Research Use Only. Diagnostic or therapeutic applications are strictly forbidden. Products shall not be used for resale or transfer to third parties either as a stand-alone product or as a manufacture component of another product without written consent of EXBIO Praha, a.s.  
EXBIO Praha, a.s. will not be held responsible for patent infringement or any other violations of intellectual property rights that may occur with the use of the products. Orders for all products are accepted subject to the Term and Conditions available at [www.exbio.cz](http://www.exbio.cz).  
EXBIO, EXBIO Logo, and all other trademarks are property of EXBIO Praha, a.s.

DyLight™ is a trademark of Thermo Fisher Scientific Inc. and its subsidiaries.

This product is provided under an intellectual property license from Thermo Fisher Scientific. The transfer of this product is conditioned on the buyer using the purchased product solely in research conducted by the buyer, excluding contract research or any fee for service research, and the buyer must not (1) use this product or its components for (a) diagnostic, therapeutic or prophylactic purposes; (b) testing, analysis or screening services, or information in return for compensation on a per-test basis; or (c) manufacturing or quality assurance or quality control, and/or (2) sell or transfer this product or its components for resale, whether or not resold for use in research. For information on purchasing a license to this product for purposes other than as described above, contact Thermo Fisher Scientific, 5781 Van Allen Way, Carlsbad, CA 92008 USA or [outlicensing@thermofisher.com](mailto:outlicensing@thermofisher.com).



EXBIO Praha, a.s. • Nad Safinou II 341 • 252 50 Vestec • Czech Republic  
[info@exbio.cz](mailto:info@exbio.cz) • [technical@exbio.cz](mailto:technical@exbio.cz) • [orders@exbio.cz](mailto:orders@exbio.cz) • [www.exbio.cz](http://www.exbio.cz)